



iRES⁺
all for a smile

iRES SAGL, a dynamic and flexible company sensitive to the needs of Professionals, offers a complete range of products for oral surgery.

iRES combines practical experience and **scientific knowledge** to **facilitate procedures and improve performance**.

High swiss quality meets **advanced technology** to provide an **innovative product concept**.

iRES⁺
all for a smile

TAKE CARE
OF **YOUR**
SMILE



iRES⁺
all for a smile



IRES SAGL

Piazzale Roncaa 4 6850 Mendrisio [CH]
<https://www.en.ires.dental/>

Thanks to **implants**, you rediscovered the **pleasure of smiling** and the serenity of showing yourself in public. In order to preserve this **quality of life**, it is important to take care of your own implant in the long term. As for a natural tooth, the implant requires **regular checkups** and an **accurate oral hygiene**.

Let's see together which are the main aspects to be remembered during the post-implantation therapy.

WHAT TO DO IMMEDIATELY AFTER THE OPERATION

SWELLING

After the operation, use ice outside the treated area, in order to avoid pain and swelling.

PAIN

Take medicines according to your dentist's prescriptions.

ORAL HYGIENE

In order to avoid bleeding in the oral cavity, it is better not to rinse using mouthwashes during the first few hours after the operation.

FOOD

During the entire recovery period, the implant can be covered by a "healing screw" or by a temporary prosthesis. To ensure a safe healing, avoid chewing hard foods on the side of the implant positioning.

WHAT TO AVOID

LYING DOWN

For the first few days we recommend to **sit** in order to keep your **head lifted** if compared with the body, day and night. Furthermore, we do not recommend to lie down on the side on which the implant was inserted.

DRIVING IMMEDIATELY AFTER THE OPERATION

The **anesthetics** could alter the ability to drive: ask the dentist for indications.

USING THE TOOTHBRUSH ON THE TREATED AREA

The dentist will suggest you the **best way to clean the treated area** without using the **toothbrush**, until the stitches are removed.

EATING AFTER THE OPERATION

Wait up to the **depletion** of the **anesthetic effect** before eating.

BAD HABITS AND SPORTS

Avoid drinking **alcohol, nicotine, coffee, tea, sports and physical efforts** for the first few days after the operation.

WHEN YOU SHOULD CONSULT THE DENTIST

The implant insertion has become a routine procedure; however the side effects, although they are very infrequent, can occur. If you may notice one of the following symptoms, or in cases of doubt, call immediately your dentist.

DAY OF SURGERY

Throbbing **pain**

Long lasting **bleeding**

Slackening of the temporary **tooth**

AFTER 12 HOURS

Numbness sensation in the treated area

THE FOLLOWING DAYS

Pain or **swelling** on the treated area

AN IMPLANT IS LIKE A NATURAL TOOTH: TAKE CARE OF IT!

Regular **checkups** ensure the **maintenance** of the functionalities and the aesthetic result of the implant, for the patient's total **safety**.

The dentist will show you the best way to **take care of the implant**: traditional or electric toothbrush and dental floss will be the allies of your smile!

